



**OWEN MUMFORD**  
Making a World of Difference

**HEAD OFFICE**

Owen Mumford Ltd  
Brook Hill, Woodstock  
Oxfordshire OX20 1TU, United Kingdom  
**T** : +44 (0)1993 812021  
**F** : +44 (0)1993 813466  
**E** : info@owenmumford.co.uk

**UK / IRELAND SALES & MARKETING**

Owen Mumford Ltd  
Primsdown Industrial Estate  
Worcester Road, Chipping Norton  
Oxfordshire OX7 5XP, United Kingdom  
**T** : +44 (0)1608 645555  
**F** : +44 (0)1608 646546  
**E** : info@owenmumford.co.uk

**THE AMERICAS**

**T** : +1 800-421-6936  
**F** : +1 770-977-2866  
**E** : info@owenmumfordinc.com

**FRANCE**

**T** : +33 (0)2 32 51 88 70  
**F** : +33 (0)2 32 21 96 95  
**E** : contact@owenmumford-fr.com

**GERMANY**

**T** : +49 (0)6026/97750  
**F** : +49 (0)6026/977529  
**E** : info@owen-mumford.de

**CHINA**

**T** : +86 (0)21 6032 3528  
**F** : +86 (0)21 6032 3782  
**E** : info@owenmumford.co.uk

[owenmumford.com](http://owenmumford.com)

**OUR ACCREDITATIONS INCLUDE:**

- ISO13485 Certification
- Medical Device Directive 93/42/EEC
- CMDCAS 13485 (Canadian Health Authority)
- Environmental Management Systems ISO14001



Reference:

1. Shah V, et al. Evaluation of a New Lancet Device on Pain Response and Success of Procedure in Term Neonates. *Arch Pediatr Adolesc Med* 2003; **157**: 1075-1078

AT/UTH-HP-LTMCC/1114/7



Tiny feet deserve  
a Tiny Touch™

OWEN MUMFORD

**Unistik® TinyTouch™**

Single Use Heel Incision Device



More reason to smile

[owenmumford.com](http://owenmumford.com)

# Unistik® TinyTouch™

Tiny feet deserve a TinyTouch™

Neonatal capillary blood testing presents many challenges for healthcare professionals – producing adequate sample volume in a single incision, yet minimising any discomfort to the baby and distress to the family.

Unistik® TinyTouch™ features a motion blade designed to deliver a smooth, precise incision to help you get it right the first time.

With penetration depths of 0.85mm for preterm neonates and 1.00mm for full term neonates our Unistik® TinyTouch™ has been designed to reach superficial blood vessels without penetrating deep enough to reach deep dermal pain fibers. This type of device results in less pain for your tiny patients.<sup>1</sup>

## Activation button.

After use, the activation button remains depressed and the **blade retracts instantly and permanently**, preventing re-use or injury.



## Sterility cap.


The easy-grip, quick-release pull tab reduces prep time and helps to **prevent accidental incisions**, while keeping the blade sterile until ready for use.

## Alignment guide.

Clearly visible from underneath and either side of the device, the alignment guide **enables precise placement**.



## How to use Unistik® TinyTouch™

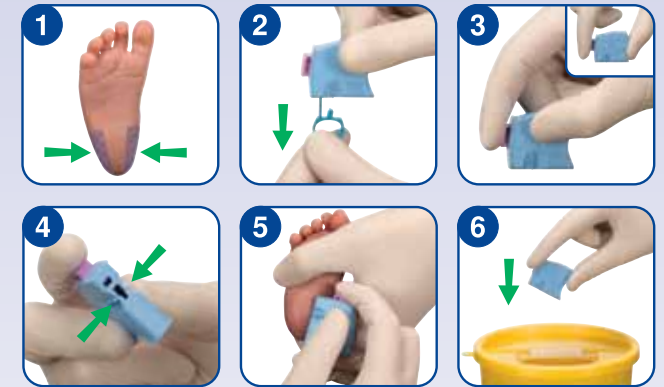

## Instructions for Use

In accordance to your institution's guidelines, gently warm the infant's heel and cleanse the outer side of the heel where you will perform the incision (1).

Remove the sterility tab by pulling straight off (2).

There are two ways to grip the device. The preferred position is with thumb and finger gripping on the wider side. Alternatively, you may grip with your thumb against the narrow side (3).

Using the alignment guide (4), align the blade slot with the incision site (5), then activate the button. Blade will permanently retract after use. Dispose of in a suitable sharps container (6).

**To express blood, gently apply pressure at the top and bottom of the incision site.**

## Product range



### Premie

Depth: 0.85mm  
Length: 1.75mm  
Box of 50: AT2004



### Full Term

Depth: 1.00mm  
Length: 2.50mm  
Box of 50: AT2005