**Side fire single-use safety lancet. Instructions for use.**

Indications for use: The single-use safety lancets are hand-held disposable devices intended to be used to achieve a controlled skin puncture, typically on the fingertip, to obtain a capillary blood specimen. Only single-use disposable blood lancets should be used for assisted blood draws by healthcare providers or at healthcare provision sites.

Wash hands thoroughly before and after using the device. Disinfect the sample site and use personal protective equipment according to your institution guidelines. 1. Twist off and discard lancet cap. **DO NOT PULL.** 2. Press platform firmly against sample site and press release button. The lancet will automatically retract into body of device. **Dispose of the entire device in a suitable sharps container.**

If any serious incident occurs with the use of this device please report to Owen Mumford Limited and your local competent authority.

The cap is the sterile barrier, do not use if cap previously removed. **For use only on a single patient. Discard the entire device after use.** Warning: Not intended for more than one use. Do not use on more than one patient. Improper use of blood lancets can increase the risk of inadvertent transmission of bloodborne pathogens, particularly in settings where multiple patients are tested. The lancet is designed with a spring to automatically retract the needle into the body of the device after use to prevent injury. Patients younger than 18 years old should be supervised by adults when using the subject device, or adults should perform the procedure on patients younger than 18 years old. Risk of injury. Before using the lancing device for the first time on children, ask your healthcare professional to ensure proper penetration depth.

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Owen Mumford GmbH,
**CH REP** Freyr Life Sciences GmbH,