

Instructions for Use for your Aidaptus Autoinjector

A Before you start

Remove from fridge 30 minutes before you are ready to inject.


Do not remove the cap until you are ready to inject.

Inspect the medicine through the viewing window. It should be clear and colourless.

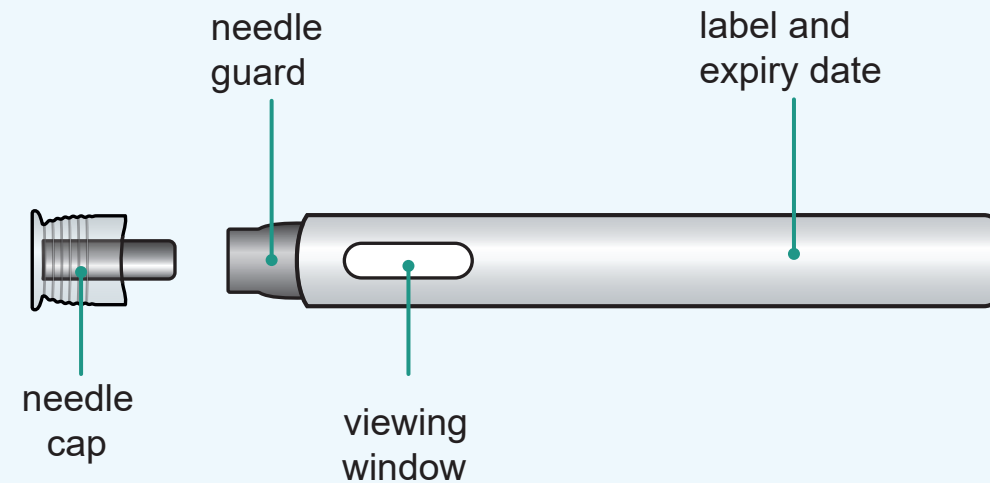
 **Do not use if the medicine looks coloured, or if it has any particles in it.**

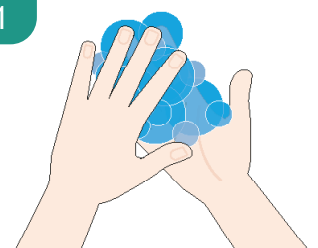
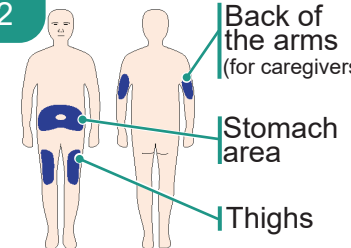
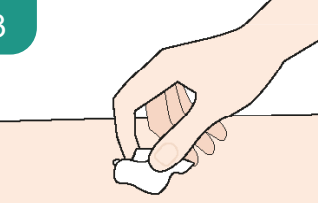
Check the name of the medicine on the Autoinjector label to make sure it is the correct one.

Check the expiry date on the Autoinjector label.

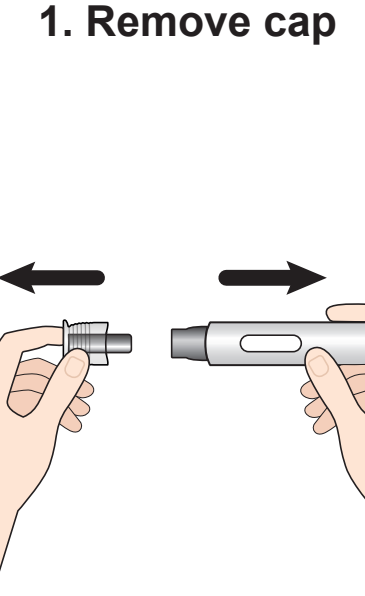
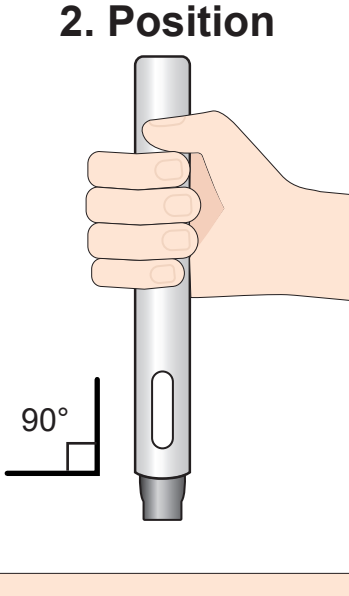
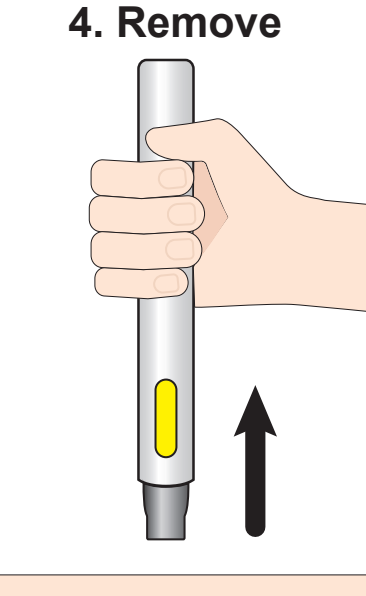

 **Do not use if it is past the expiry date.**

B Preparation



<p>1</p> 	<p>2</p>  <p>Back of the arms (for caregivers) Stomach area Thighs</p>
<p>Wash your hands with soap and water.</p>	<p>Choose your injection site</p> <p>Do not inject into areas that are red, hard, bruised, or tender.</p>
<p>3</p> 	<p>Wipe the skin at the injection site with an alcohol swab.</p>

C Please read the instructions carefully before injecting the medicine

<p>1. Remove cap</p> 	<p>2. Position</p>  <p>90°</p>	<p>3. Push <u>down</u> and keep holding <u>down</u></p>			<p>4. Remove</p> 	<p>5. Throw away</p> 
<p>Pull the needle cap straight off the autoinjector.</p> <p>Throw away needle cap in the sharps container.</p>	<p>Hold the autoinjector at 90 degrees to the injection site.</p> <p>Keep the viewing window facing towards you, if possible.</p>	<p>Push down and keep HOLDING down the autoinjector to start the injection.</p> <p>You will hear Click 1 immediately when the injection starts.</p>	<p>Do NOT remove.</p> <p>You will hear Click 2. This could take up to 10 seconds.</p> <p>The viewing window will turn yellow.</p>	<p>After Click 2, SLOWLY count 1..2..3..4..5 to make sure all of the medication has been injected.</p> <p>Check the viewing window is yellow.</p>	<p>Remove the autoinjector. The needle will be safely covered.</p>	<p>Put the used autoinjector into an approved sharps container.</p>