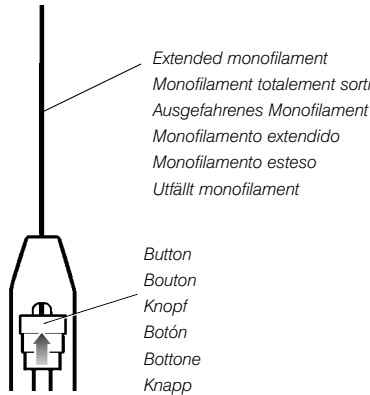
1

EN Assessing protective touch/pressure sensation

Monofilament Test

1. Prepare patient according to guidelines.
2. Fully extend the monofilament by sliding the button to the end of the device until a click is heard.
3. Wipe the end of the monofilament with an alcohol wipe or antiseptic solution.
4. Press the monofilament at a 90° angle to the skin surface and increase the pressure until it bows.
5. Hold in position for 1-2 seconds.
6. Suggested sites to be tested are shown in the diagram and should be tested at random.

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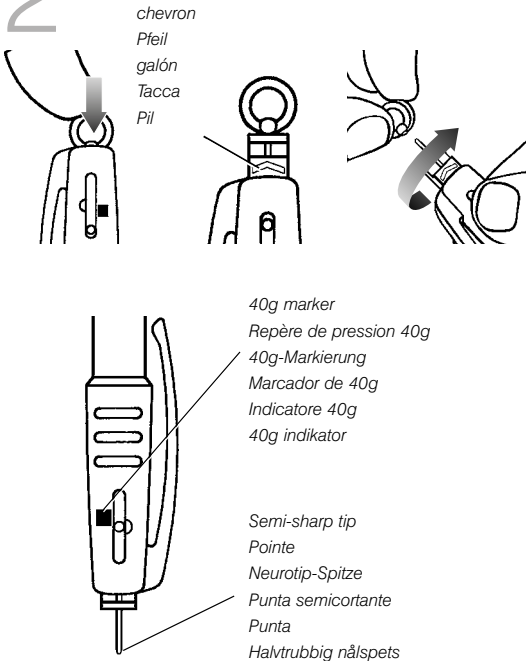
2

Assessing protective pain/ sharpness sensation

Calibrated Neurotip Test

This is a calibrated pain/sharpness test. The spring mechanism is calibrated to exert a force of 40g, which will help to identify patients with a loss of pain sensation.

1. Take an **unused** Neurotip and hold by the cap.
2. Press Neurotip firmly down into the Neurotip carrier of the Neuropen as far as it will go and clicks into position. The Neurotip is correctly positioned when only **one chevron** can be seen on the Neurotip.
3. To expose the semi-sharp tip, place thumb gently over the Neurotip pressure gauge and remove Neurotip cap by twisting and pulling onwards.
4. Press the Neurotip against the test surface at a 90° angle. Hold in place for 1-2 seconds before removing, taking care to **depress within the 40g marked white area**.
5. Ask the patient if they detect a sharp sensation.
6. **IMPORTANT: Remove and discard the used Neurotip in an appropriate sharps container.**

CAUTION: Do not use if Neurotip cap has previously been removed.

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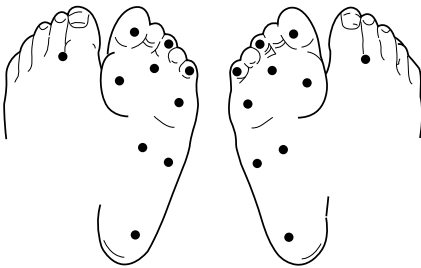
3

Comparative sensory testing

Neuropen makes it possible to easily assess whether the patient can distinguish between a touch stimulus (monofilament) and a sharp stimulus (Neurotip).

Randomly test the patient with either the Neurotip or monofilament by inverting the pen and ask the patient to identify which stimulus they feel.

A normal individual will easily discriminate between these sensations – an inability to distinguish identifies patients at risk of peripheral neuropathy.

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