95mm

FOLD

ENGLISH:

Product description: Unistik® VacuFlip is a single use, sterile, blood collection phlebotomy needle with safety mechanism to prevent needlestick injuries.

Intended use: The Unistik® VacuFlip blood collection needle is intended to obtain blood specimens from patients into blood collection tubes or blood culture bottles. The device is intended to be used by appropriately trained healthcare professionals.

Blood collection safety needle with pre-attached holder

For blood collection

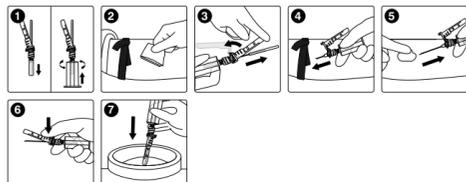
- 1. Tear off the blister. While holding the needle guard and the needle cap, screw holder in a clockwise direction to ensure needle is firmly fitted onto the holder.
2. Prepare venipuncture site in accordance with your facility's procedure.
3. Flip the needle guard back towards the holder and remove needle sheath. DO NOT twist or rotate the needle guard.
4. Perform venipuncture and collect the blood specimen in accordance with your facility's procedure. Hold the tube in place until blood stops flowing.
5. After completion of blood collection, gently remove needle from vein. Cover the puncture site with a sterile gauze and apply pressure.
6. Flip the needle guard forward with thumb and lock the needle cannula until an audible click is heard indicating the needle is completely guarded and locked to avoid possible needlestick injury.
7. Dispose of all used materials as per your facility's policy.

Blood collection safety needle

For blood collection

Note: Use the Unistik® Holder for blood collection

- 1. Tear off the blister. Twist and remove Luer sheath and screw holder onto needle. While holding the needle guard and the needle cap, screw holder in a clockwise direction to ensure needle is firmly fitted onto the holder.
2. Prepare venipuncture site in accordance with your facility's procedure.
3. Flip the needle guard back towards the holder and remove needle sheath. DO NOT twist or rotate the needle guard.
4. Perform venipuncture and collect the blood specimen in accordance with your facility's procedure. Hold the tube in place until blood stops flowing.
5. After completion of blood collection, gently remove needle from vein. Cover the puncture site with a sterile gauze and apply pressure.
6. Flip the needle guard forward with thumb and lock the needle cannula until an audible click is heard indicating the needle is completely guarded and locked to avoid possible needlestick injury.
7. Dispose of all used materials as per your facility's policy.

General precautions

Handled by appropriately trained healthcare professionals. Always follow Universal Precautions. Gloves should be worn throughout venipuncture to minimize exposure hazard. Handle all biologic samples and blood collection "sharps" (needles, Luer adapters and blood collection sets) in accordance with your facility's policy. Obtain appropriate medical attention in the event of any exposure to biologic samples (e.g. through a puncture injury) since samples may transmit viral hepatitis, HIV/AIDS or other infectious diseases. Refrain from removing the rubber sleeve, or forcefully pulling the safety guard, as such actions might damage the integrity of the product. Do not forcefully release or re-activate the safety mechanism after it has been activated. Exercise caution when activating the safety mechanisms against a hard surface, as this may cause blood splatter and breakage of the product. Keep hands behind needle at all times during use and disposal of the device. Do not bend the needle. Do not attempt to engage needle guard by pressing against a hard surface, as this may result in blood splatter. Do not reuse. A reuse of the product may cause harmful infections, injury or death. Do not use if the package is opened or damaged, the needlestick is defective or the safety guard is not firmly assembled. Do not remove needle from holder after use. Dispose of the needle and holder as one unit in biohazard containers approved for their disposal. If any serious incident occurs with the use of this device, please report to Owen Mumford Limited and your local competent authority.

Production batch no. and expiration date

Production batch # and expiry date on the seal of package. Shelf life: 5 years. Needles expire on the last day of the month and year indicated.

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

FOLD

Symbol and Mark Key

Table with 2 columns: Symbol and Mark Key, and Description. Symbols include: Non-pyrogenic, Sterile EO, Caution, Single Use, Not made with natural rubber latex, Not made with DEHP, Medical Device (MD), Keep Away from Sunlight, Information (i), Batch Number (LOT), Date of Manufacture, Manufacturer, Use by Date, EU Authorized Representative (EC REP), Rx only, and CE mark.

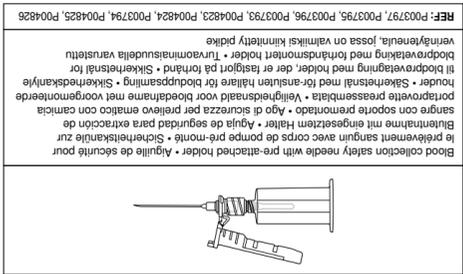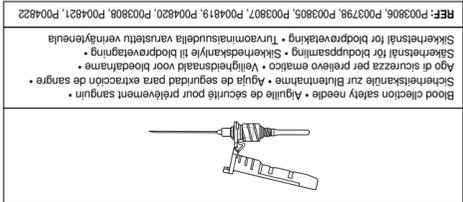OWEN MUMFORD

Instructions for use • Mode d'emploi • Gebrauchsanweisung • Käyttöohje • Instruções de uso • Istruzioni per l'uso • Gebruiksaanwijzing • Bruksanvisning • Brugsanvisning • Bruksanvisning • Käyttöohje

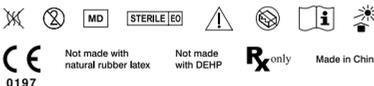
Unistik® VacuFlip

Manufactured by Yangzhou Medline Industry Co., Ltd., No. 106 Jinshan Road, Economic Development Zone, Yangzhou 225009 Jiangsu, China

Manufactured for Owen Mumford Ltd., Brook Hill, Woodstock, Oxfordshire OX20 1TU, United Kingdom T : +44 (0)1993 812021 E : info@owenmumford.com

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