**ENGLISH :**

**Single-use heelstick safety lancet: instructions for use**

**[Structure]** The Heelstick safety lancets consist of blade, blade base, top head, bottom and pushing button.

**[Performance]** The device is sterilized by gamma irradiation.

**[Intended use]** The heelstick safety lancets are single-use devices which are designed to collect a capillary blood sample from the heel of infants.

**[Contraindications]** None.

**[Instructions]**

1. Select the test site. Clean the site with an antiseptic swab and allow to air dry.
2. Open the blister package and take out the lancet.
3. Place the lancet against the baby's heel and activate by pressing inwards on the white button. Then collect the blood sample.
4. Discard the blister and the used heelstick safety lancet in appropriate container.

**[Cautions]**

1. The individual blister pack is the sterile barrier, do not use if the individual package is open or damaged as infection may occur.
2. Not suitable for home use by lay person.
3. The reuse of this single-use device will affect safety, performance and effectiveness, and may cause infection.
4. In the event of any serious adverse event, please report to Owen Mumford and your local competent authority.
5. Only use on area indicated and appropriate birthweight or age, according to your institution's instructions.
6. Only single-use disposable blood lancets should be used for assisted blood draws by healthcare professionals or at healthcare provision sites.
7. Although the products have undergone various biological tests, and the test results are all in line with the standard requirements, allergic or irritating reactions may occur due to individual differences.

**[Storage]** Store in a well ventilated room. Keep away from light, moisture and corrosive gases.

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Owen Mumford Ltd.,
Owen Mumford GmbH,