## ENGLISH: Instructions for use

Please refer to your blood glucose monitoring system for instructions on how to obtain a blood glucose reading.

### AUTOLET PLUS COMPONENTS (REFER TO ILLUSTRATIONS)

- Depth adjuster
- Depth setting indicator window
- Activation button
- Cocking/lancet release slider
- Lancet holder
- Lancing device cap
- Lancet cap
- Lancet
- Clear cap (for alternative site testing)

### USING AUTOLET PLUS

- Remove the lancing device cap (f) by snapping it off at an angle.
- Insert a new lancet (h) into the lancet holder (e). This action may cock the device.
- Hold the lancing device firmly in one hand, use your other hand to twist off the lancet cap (g).

- Place the lancing device cap (f) back on the device with the depth setting indicator window (b) aligned with the activation button (c). Be careful not to touch the exposed needle of the lancet.

- Set the depth adjuster (a). Autolet Plus offers 9 depth settings. Level 1 is the shallowest depth; level 9 is the deepest.

- Priming the device (d). Pull back on the grey slider until it clicks. (You may have already cocked/primed the device in step 2.) You are now ready to perform a blood glucose test.

- To lance your finger hold the lancing device firmly against the side of your fingertip. Press the activation button (c). Apply gentle pressure near the puncture site if needed, until a blood drop appears.

Refer to your User's Manual for further information on testing with your blood glucose monitoring system.

- To release the used lancet, first remove the lancing device cap (f), push the lancet release slider forward/downward (d).

If the lancet does not detach, push the slider forward and carefully hold the sides of the exposed lancet. Firmly pull the lancet away from the device.

The lancet should be removed and disposed in accordance with local waste regulations.

### ALTERNATE SITE TESTING (AST)

Refer to your test strip's instructions for more information on test sites other than your fingertips.

Choose a penetration site on a soft, fleshy area of your arm, or other recommended site, away from bone and free of visible veins, moles, bones and tendons.

Use the clear cap for alternate site testing. **This package does NOT include an alternate site cap.**

**If you require one please contact your appropriate Owen Mumford office. Contact numbers can be found at the bottom of these instructions. If you do not have an Owen Mumford office in your country please contact Owen Mumford's Head office.**

Follow step 1 – 3 as above.

- Replace the normal lancing device cap (f) with the clear alternate site cap. The alternate site testing cap has a fixed depth setting.

Follow step 6 as above.

- Rub the test site for a few seconds until it is warm. Hold the lancing device firmly against the test site. Press the activation button (c). Continue to hold the lancing device against the test site. Gradually increase the pressure for a few seconds.

- Lift the lancing device straight up; be careful not to smear the blood sample.

### CLEANING

**Important:** Do not immerse the body of the lancing device in water or any other liquid.

Use isopropyl alcohol or soap and water to wipe the outside of the lancing device. If you wish, remove the cap, wash it in warm water and rinse well. Store lancing device in carry case or in a clean and dry environment.

### SAFETY INFORMATION

Autolet Plus is intended for self-monitoring by a single user only and must not be used in multi-patient clinical environments.

Remove lancet after use. Do not store the device with the lancet inserted.

### WARRANTY INFORMATION

The warranty offered by Owen Mumford Limited ("we") in relation to this product is set out at [owenmumford.com](http://owenmumford.com). PLEASE READ THIS WARRANTY IN FULL. Your rights under the laws of your country are not affected.

For date of manufacture please see dispenser box.

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### RECOMMENDED UNILET LANCETS / LANCETTES UNILET RECOMMANDÉES

Multiple use devices should not be used for assisted blood draws by healthcare providers or at healthcare provision sites and should never be shared with anyone else, even a family member. Do not use if cap has been previously removed. Due to the risk of cross infection, this single use device is for self-monitoring by a single user only. Reuse of this single use device will affect safety, performance and effectiveness, and exposes users to unnecessary risks. Please note that not all products are available in all countries.

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Please check contents of the package as listed on the outside of the box.

Autolet®, Unilet® and Comfort Zone Technology® are registered trademarks of Owen Mumford Ltd. International Patents Pending.

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Autolet® Plus and Unilet® ComforTouch™ made in England for Owen Mumford. Unilet® Eco/Excelite made in China, packaged by Owen Mumford.

**HEAD OFFICE**  
Owen Mumford Ltd  
Brook Hill, Woodstock  
Oxfordshire OX20 1TU  
United Kingdom  
T : +44 (0)1993 812021  
E : [info@owenmumford.co.uk](mailto:info@owenmumford.co.uk)

**UK / IRELAND SALES & MARKETING**  
T : +44 (0)8000 320439  
E : [info@owenmumford.co.uk](mailto:info@owenmumford.co.uk)

**THE AMERICAS**  
T : +1 800-421-6936  
E : [info@owenmumfordinc.com](mailto:info@owenmumfordinc.com)

**FRANCE**  
T : +33 (0)2 32 51 88 70  
E : [contact@owenmumford-fr.com](mailto:contact@owenmumford-fr.com)

**GERMANY**  
T : +49 (0)6026/97750  
E : [info@owen-mumford.de](mailto:info@owen-mumford.de)

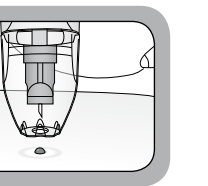
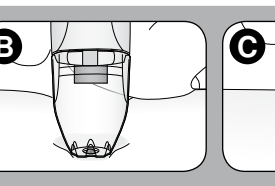
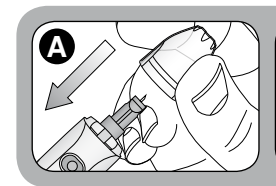
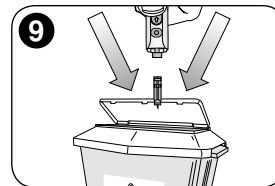
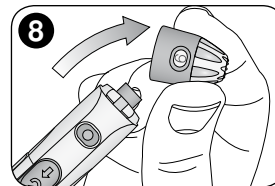
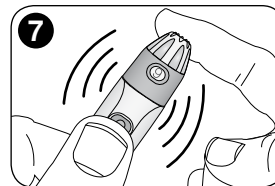
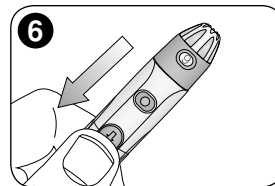
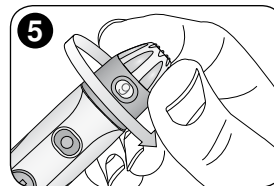
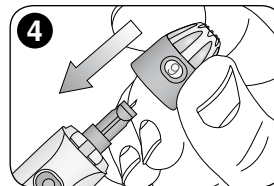
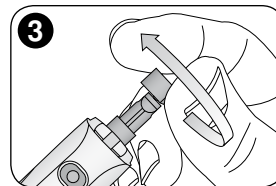
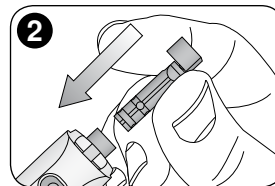
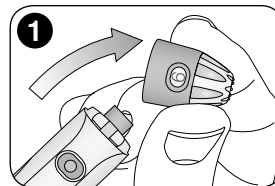
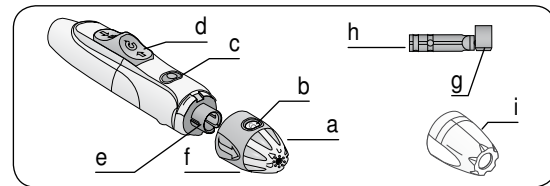
**CHINA**  
T : +86 (0)21 6032 3528  
E : [chinfo@owenmumford.com](mailto:chinfo@owenmumford.com)

**MALAYSIA**  
T : +60 (0)7 553 9617  
E : [myinfo@owenmumford.com](mailto:myinfo@owenmumford.com)

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