Autopen[®] 24

Instructions for Use

For intended use only.

Autopen should be used only after consultation with your healthcare professional.

Autopen 24 is an easy-to-use insulin injection pen, designed for use with 3.0ml sanofi insulin cartridges. Compatible with all pen needles*.

PLEASE FOLLOW THESE STEP-BY-STEP INSTRUCTIONS CAREFULLY BEFORE USING THE AUTOPEN 24.

Failure to follow these instructions may result in an inaccurate dose.

Please refer to the numbered pictures in the pull out section which accompany the following steps for use.

LIST OF COMPONENTS

- a Dose selector
- b Pen body
- c Release button
- d Internal plunger
- e Cartridge
- f Cartridge holder
- g Pen needle & protective caps
- h Pen cap
- i Release button extension – simply clips onto release button for improved grip
- j Dose selector adapter – fits onto dose selector for easier dialling

*Data on file. Content correct at time of publication.

Recommended for use with:

Unifine[®] Pentips[®]

PEN NEEDLES

Do not use if protective seal is damaged.

To order more Unifine Pentips, please contact your local Owen Mumford office.

LOADING A CARTRIDGE

1. Pull off the pen cap and unscrew the pen body from the cartridge holder.
2. **Check to ensure that the internal plunger is fully retracted (fig. 2). If the internal plunger is extended, please refer to step 12 (fig. 12) to see how to wind it back in.**
3. Load the insulin cartridge into the cartridge holder, metal cap in first.
4. Screw the cartridge holder and pen body firmly together.

PRIMING YOUR AUTOPEN 24

5. Peel the protective film from a new pen needle. Screw the pen needle **straight** onto the cartridge holder (fig 5). Remove the outer protective cap and needle cap from the pen needle.

***If the cartridge contains longer acting NPH (Basal) insulin, turn the pen up and down at least 10 times as per cartridge instructions. Then attach the needle to the pen cartridge holder.**

Steps 6 & 7 must be followed before each injection.

6. It is important to prime the Autopen 24 before use to remove any air that may be inside the needle. When using a new cartridge dial 8 units on the dose selector (figs.6a/6b).

7. Hold the Autopen 24 with the pen needle pointing up. Push and hold the release button towards the needle until the arrow ► on the pen body returns to the start line ◼ on the dose selector. On subsequent occasions you need only dial to 2 units.

Repeat dialling and releasing 2 units until insulin appears at the needle tip. (fig.7a/7b).

The Autopen 24 is now ready for use.

If the start line ◼ on the dose selector does not return to the arrow ► on the pen body, please refer to the Priming Advice information found on the reverse of the pictorial instructions.

INJECTING YOUR DOSE

8. Make sure the arrow ► on the pen body points to the start line ◼ on the dose selector.

Dial up the number of units required.

Never attempt to dial back the dose selector, as doing so may damage the Autopen 24 and result in inaccurate dosing.

If at any time the dose has been over dialled, we recommend that the incorrect dose is fully expelled and the required dose is redialled.

Make sure before injecting that the arrow ► points towards the correct number of units on the dose selector.

For example, figures 8a and 8b show the correct position for 20 units of insulin delivery.

9. Insert the needle using the injection technique recommended by your healthcare professional. Push the release button towards the pen needle and hold it in place until the start line on the dose selector returns to the arrow on the pen body ► ◼ and continue holding the release button down for 10 seconds. Then remove the needle from the skin.

If the dose selector stops before the start line ◼ returns to the arrow ►, this indicates that you have not received a full dose. The dose selector will show the number of units that must be injected to complete the insulin dose.

REMOVING THE NEEDLE

10. Reattach the white outer protective needle cap and unscrew the pen needle. Always check your pen needle has been removed. Replace the pen cap onto the Autopen 24. Dispose of used pen needles carefully in accordance with the advice given by your healthcare professional or local regulations.

CHANGING THE CARTRIDGE

11. Remove the needle, unscrew the cartridge holder from the pen body and remove the cartridge.

12. Rewind the internal plunger by holding the grey lower part of the pen body between the thumb and finger. Turn the pen body until the internal plunger is fully retracted. Then return to step 3 to load the next cartridge.

SAFETY PRECAUTIONS

- Autopen 24 should be used only after consultation with your healthcare professional.
- Autopen 24 is only for use with 3.0ml sanofi insulin cartridges.
- Read and follow the manufacturer's instructions provided with your insulin cartridges. Please store your insulin at the temperature recommended by the manufacturer.
- Before each injection check that the Autopen 24 contains the correct insulin type, as prescribed by your healthcare professional.
- Always inspect your Autopen 24 for any signs of damage before use. Do not use your Autopen 24 if dropped onto a hard surface as this may result in damage to the device.
- Always ensure that Autopen 24 has been primed in accordance with the instructions detailed in steps 6 & 7. Failure to follow the exact priming procedures could result in an inaccurate dose being delivered.
- The needle should be removed and safely disposed of immediately after each injection. If the needle is left on it may become blocked and affect the dose delivered.

- If when removing the pen needle any seepage of insulin is found, it is possible that you may not have injected the full insulin dose. Do not try to make up for the shortfall in your insulin dose by giving a second injection (otherwise you will be at risk from low blood sugar). As a precaution we suggest you check your blood sugar at regular intervals and refer to your insulin instructions for use or contact your healthcare professional.
- Consult your healthcare professional should you encounter abnormal blood glucose levels.
- Consider replacing the device after 2 years of use.
- Please dispose the device following your local medical waste disposal procedure after 2 years of use.
- If any serious incident occurs with the use of this device please report to Owen Mumford Ltd and your local Competent Authority.

STORAGE AND CLEANING

- The Autopen 24 should always be stored or carried with the pen needle removed, and the device cap on. It should not be stored with a dialled dose.
- Cartridges can be carried in the Autopen 24 or as spares according to the cartridge instructions.
- Please store your insulin at the temperature recommended by the insulin manufacturer.
- To clean the Autopen 24, wipe with a damp cloth. Do not immerse in water.
- Do not use any other solutions, e.g. chemicals, alcohol wipes etc to clean your Autopen 24.
- Keep your Autopen 24 and Pen Needles out of reach of children.

Please check the contents of the package as listed on the outside of the box.

Autopen Accuracy Information:
For insulin doses of 20 units and below = ± 1 insulin unit.
For insulin doses above 20 units = $\pm 5\%$.

Autopen®, Unifine® and Pentips® are registered trademarks of Owen Mumford Ltd.

Sanofi is a trademark of the sanofi group of companies.

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Autopen®, Unifine® och Pentips® är av Owen Mumford Ltd. registrerade varumärken.