


Unistik[®] 3

Single Use Safety Lancets

1.

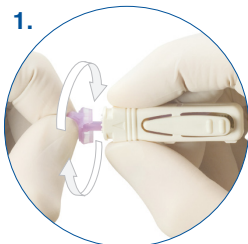

TWIST cap to “Unlock” and continue to **TWIST** until cap separates easily from lancet body.
DO NOT PULL.

2.



Choose a site on the finger as indicated by the shaded area to **POSITION** the device.

3.



Position against finger and **PRESS** release button. Lancet will automatically retract into the body of the device. **DISPOSE** of in a suitable sharps container.


U3-IFUGROMMET/OMI/0114/2/US

Unistik[®] 3

Single Use Safety Lancets



Featuring patented **Comfort Zone Technology[®]** for a more comfortable blood sample.

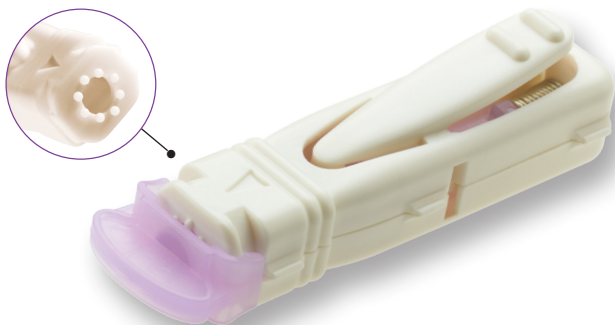

Comfort Zone Technology[®] or CZT[®].

Independent clinical data shows that Unistik[®] 3 Safety Lancets with CZT[®] reduce pain, resulting in a more comfortable experience for the patient. CZT[®] is comprised of eight raised pressure points on the patient end of the safety lancet. This patented technology sends a signal of comfort to the brain, helping to eliminate the pain associated with the fingerstick.