


# Unistik<sup>®</sup> Touch

With  
Comfort  
Zone  
Technology<sup>®</sup>

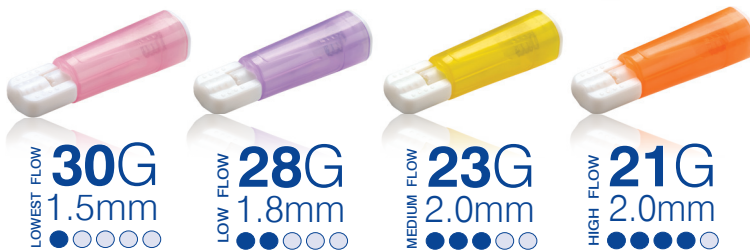

# simple



## Intuitive. Easy-to-Use. Affordable.

Unistik Touch single-use safety lancets are designed to make blood sampling easy for the healthcare professional and more comfortable for the patient. Touch activation ensures that the device requires minimal training and can be used in three easy steps.


- **Designed to be easy and intuitive, requiring minimal training**
- **Features Comfort Zone Technology for enhanced patient comfort**
- **Range of sizes to meet most testing needs and preferences**

### Take the Unistik Touch Challenge

Experience the difference Unistik Touch can make for your patients. Contact Owen Mumford for a free product trial at 1-800-421-6936 or email [info@owenmumfordinc.com](mailto:info@owenmumfordinc.com).

AS EASY AS  
**ONE.  
TWO.  
THREE.**


**1.**

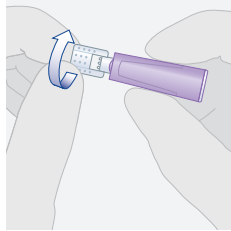

**TWIST**

Twist off and discard lancet cap. **DO NOT PULL.**


**2.**

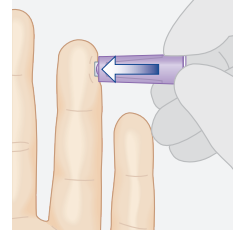

**CLICK**

Press platform firmly against sample site. Lancet will automatically retract into body of device.

**3.**

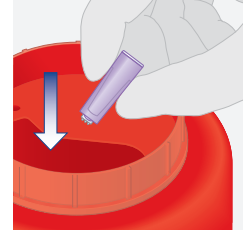

**DISPOSE**

Dispose of in a suitable sharps container.

**DESIGNED FOR  
REDUCED PAIN\***

Unistik® Touch features Owen Mumford's patented Comfort Zone Technology. When the lancet is pressed against the skin, the 8 raised dots mask the pain sensation created by the needle.

Comfort Zone Technology is based on the 'Gate Theory' and works by sending a 'comfort' message to the brain which interferes with the pain message produced during the lancing.

\*Lancet trials showed that Comfort Zone Technology® (CZT®) reduced the overall pain experienced during the blood sampling procedure when compared to devices without Comfort Zone Technology. Data on file.

**COMFORT  
ZONE  
TECHNOLOGY®**

**BROAD  
RANGE  
OF SIZES**



Lowest Flow  
**30G**  
1.5mm

Box of 100: AT1422  
Box of 200: AT1424

Low Flow  
**28G**  
1.8mm

Box of 100: AT1442  
Box of 200: AT1444

Medium Flow  
**23G**  
2.0mm

Box of 100: AT1402  
Box of 200: AT1404

High Flow  
**21G**  
2.0mm

Box of 100: AT1412  
Box of 200: AT1414



**OWEN MUMFORD**  
Making a World of Difference

**Take the Unistik Touch Challenge**

Experience the difference Unistik Touch can make for your patients. Contact Owen Mumford for a free product trial at 1-800-421-6936 or email [info@owenmumfordinc.com](mailto:info@owenmumfordinc.com).